

TOVA INSTALLATION GUIDE



1. INSTALL THE SOFTWARE

- You'll need to have administrator privileges to install the T.O.V.A.
- Insert the CD (or included USB flash drive).
 - Windows Vista/7 – When prompted, choose 'Run T.O.V.A. Setup for Windows.exe'.
 - Windows 2000/XP – If the installer doesn't run automatically, go to My Computer, 'tova8', and double-click on 'T.O.V.A. Setup for Windows'.
 - Mac OS X 10.6 – Double-click on 'T.O.V.A. Setup for Mac' disk image, then double click on 'T.O.V.A. Setup for Mac' inside the disk image window.
- Accept the defaults in the installer and the T.O.V.A. system preferences; you can always change them later.

2. SETUP THE HARDWARE


- Choose setup (A) or (B) based on your existing computer setup.



If you **DON'T** have an external display, as on a laptop, use the:

PRECISION TEST ENVIRONMENT (PTE) SETUP


- The PTE requires a reboot of your computer in order to precisely administer the T.O.V.A. test on your screen.
- In the T.O.V.A., choose "Preferences," "Session Launch," and choose the "Precision Test Environment."

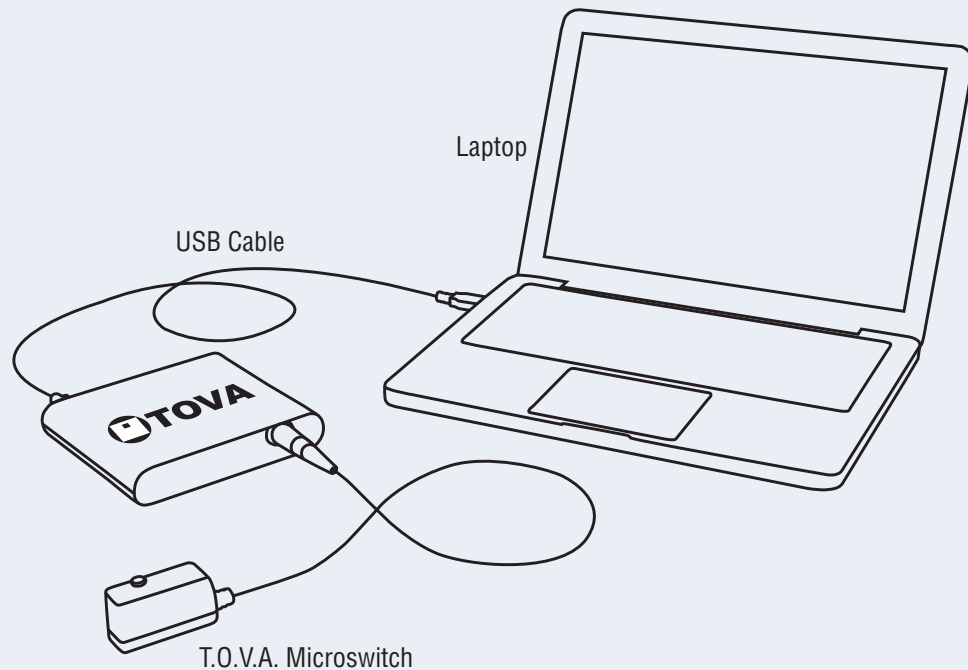

If you **DO** have an external display, as on a desktop computer, use the:

EXTERNAL AUDIO/VIDEO (EAV) SETUP

- The EAV setup allows you to give the T.O.V.A. test without rebooting.
- Optional: you can use a secondary monitor connected only to the T.O.V.A. USB device. This would, for example, allow for a laptop to perform an EAV test.

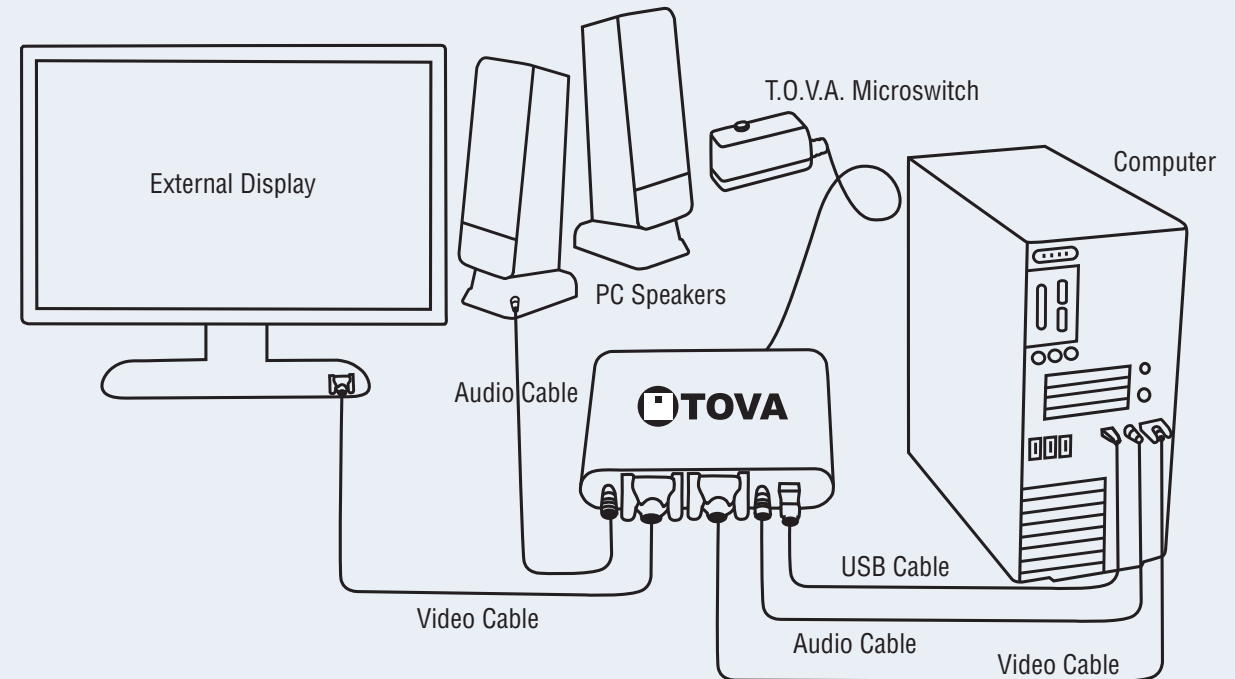

Questions? Contact T.O.V.A. Technical Support at 800.729.2886 (562.594.7700) or info@tovatest.com